

Dynamic Headrest Interface - Gas Spring

Tools needed: 5/32, 3/16 Hex keys

1. Install the Dynamic Headrest Interface Base Plate behind existing Mounting Base.
2. Install Dynamic Headrest Collar Bracket above the Pivot Bar Collar on the Vertical Adjustment Bar for the "self lock" base. On the "old style" mount, below the collar on the pivot bar.
3. Use the 4-prong knob to attach the cylinder to the base.
4. To facilitate dynamic function, pull out plunger assembly and turn 90° (1/4 turn).
NOTE: The plunger assembly will have to be removed on original style units.
5. Adjust the collar bracket to reduce movement and play.
6. To allow headrest to swing down, remove the 4-prong knob (optional).

